
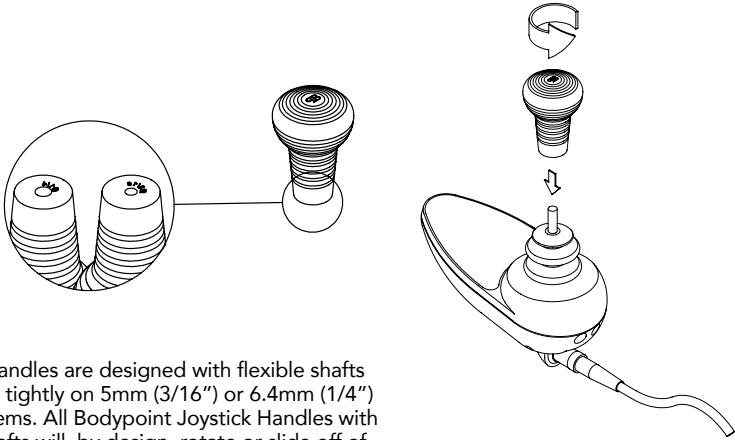
Joystick Handle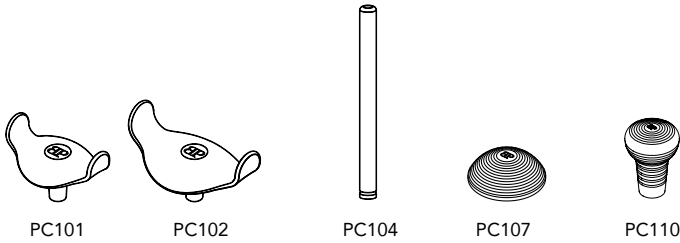


Joystick Handles are designed with flexible shafts that will fit tightly on 5mm (3/16") or 6.4mm (1/4") joystick stems. All Bodypoint Joystick Handles with flexible shafts will, by design, rotate or slide off of the steel joystick stem if subjected to undue pulling. This is to prevent damage to the joystick box. Altering the product to circumvent this feature is not recommended.

INSTALLATION INSTRUCTIONS

1. Clean the metal joystick stem to remove any dirt or oil.
2. Moisten the metal joystick stem to facilitate mounting the handle.
3. Grasp the metal joystick stem near the base with pliers to keep from pushing it into the sensitive electrical components of the joystick box.
4. Press the Joystick Handle onto the metal joystick stem with a twisting motion.

WARRANTY

This product carries a limited lifetime warranty against defects in workmanship and materials arising under normal use by the original consumer. Contact your supplier or Bodypoint for warranty claims.

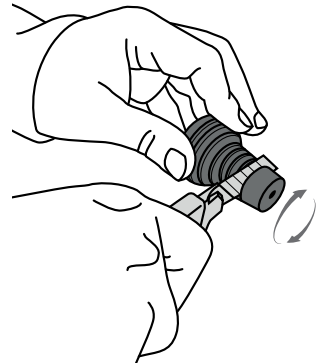PC110 MODIFICATIONS

Handle can be cut down using a sharp knife (roll through cut for best results) 28mm (1-1/8") min height

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MDSS GmbH