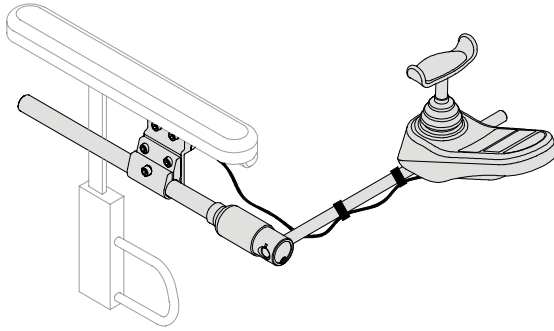
Joystick Mounting Kits

INSTALLATION AND USER'S INSTRUCTIONS

ART NOS. PC202, PC205, PC206 & PC207

These instructions provide important information for the safe use and maintenance of all Bodypoint Joystick Mounting Kits. Give these instructions to the user or their caregiver and review them to ensure that they are understood.

⚠ CAUTION! Turn off wheelchairs electrical system before installation.

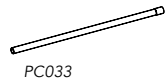
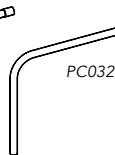
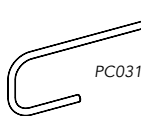
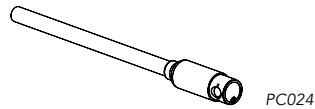
WARRANTY: This product carries a limited lifetime warranty against defects in workmanship and materials arising under normal use by the original consumer. Contact your supplier or Bodypoint for warranty claims.

For more information on Bodypoint products and a list of distributors outside the USA go to www.bodypoint.com.

PACKAGE CONTENTS

Package may contain one or more of the following:

- Joystick Mounting Clamp
- Tri-Lock™ Rotating Shaft
- Midline Arm
- Joystick Mounting Adapter
- Quick Release Lever (Optional) or a complete joystick Mounting Kit. Art PC202-, PC205-, PC206-, PC207-

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
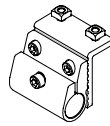
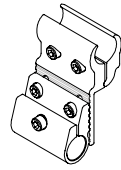
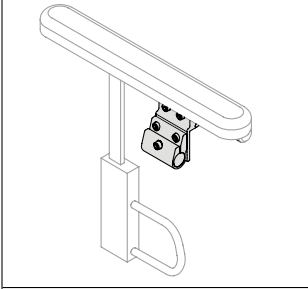
 CE

JOYSTICK MOUNTING CLAMP



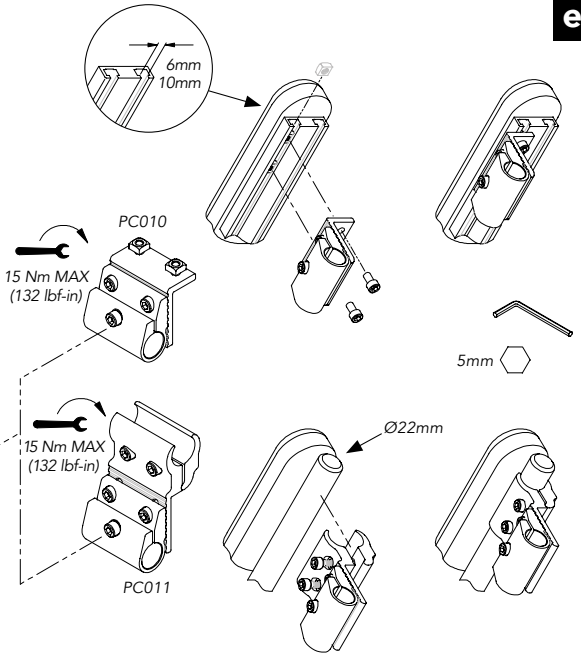
Joystick Mounting Clamps can be installed on the right or left side. Attach clamp to armrest. Position the clamp as far forward as possible and tighten.


Art. No. FA171

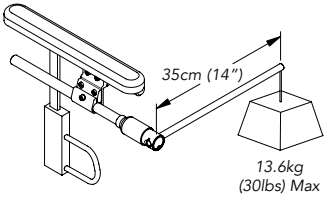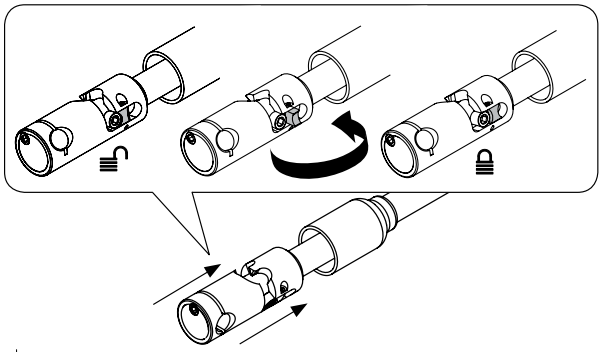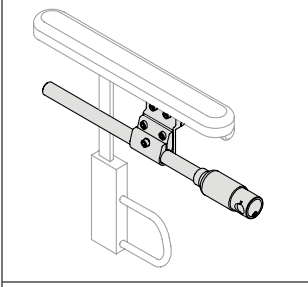TRI-LOCK™ ROTATING SHAFT

Tri-Lock Rotating Shaft is compatible with right or left side mounting. Rotate Shaft in Mounting Clamp to ensure correct positions of rotation.

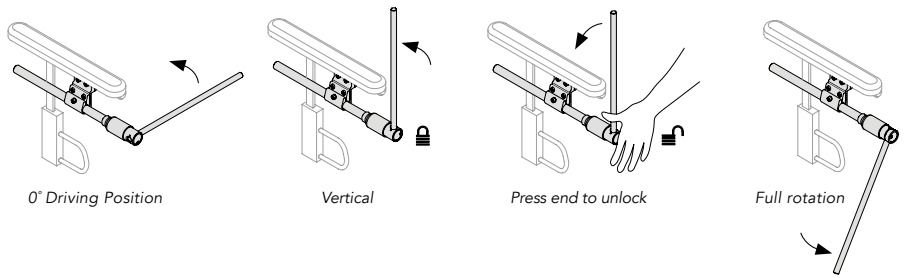The Flip-stop allows conversion from non-locking to locking. 0° horizontal position becomes locked. Press end to unlock.



0° Driving Position

Vertical


Press end to unlock

Full rotation

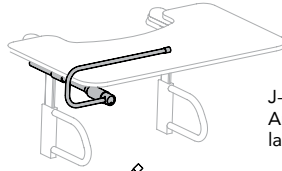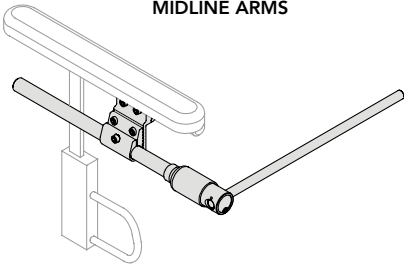MIDLINE ARMS


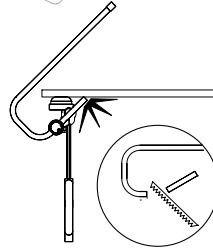
J-Shaped Midline Arm allows use over a lap tray.

NOTE: The aluminum Midline Arm can be cut to length using a hacksaw.

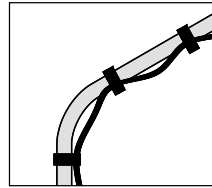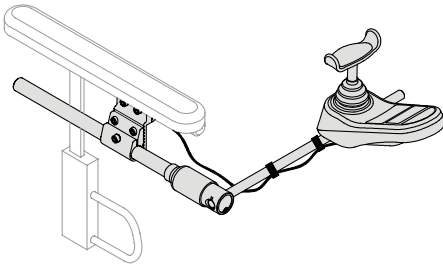
JOYSTICK MOUNTING ADAPTERS

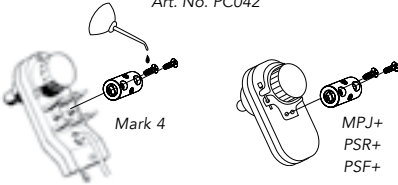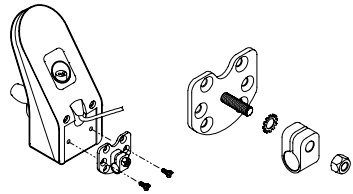
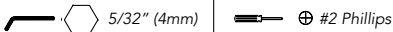
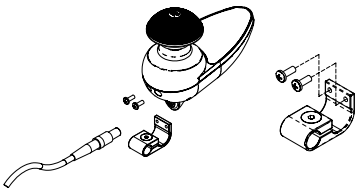
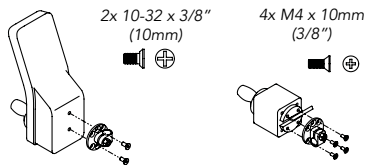
Attach loose wires with nylon ties included.

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